Prótesis de Rodilla **TREKKING[®] FB**

Folleto

Generalidades y definiciones

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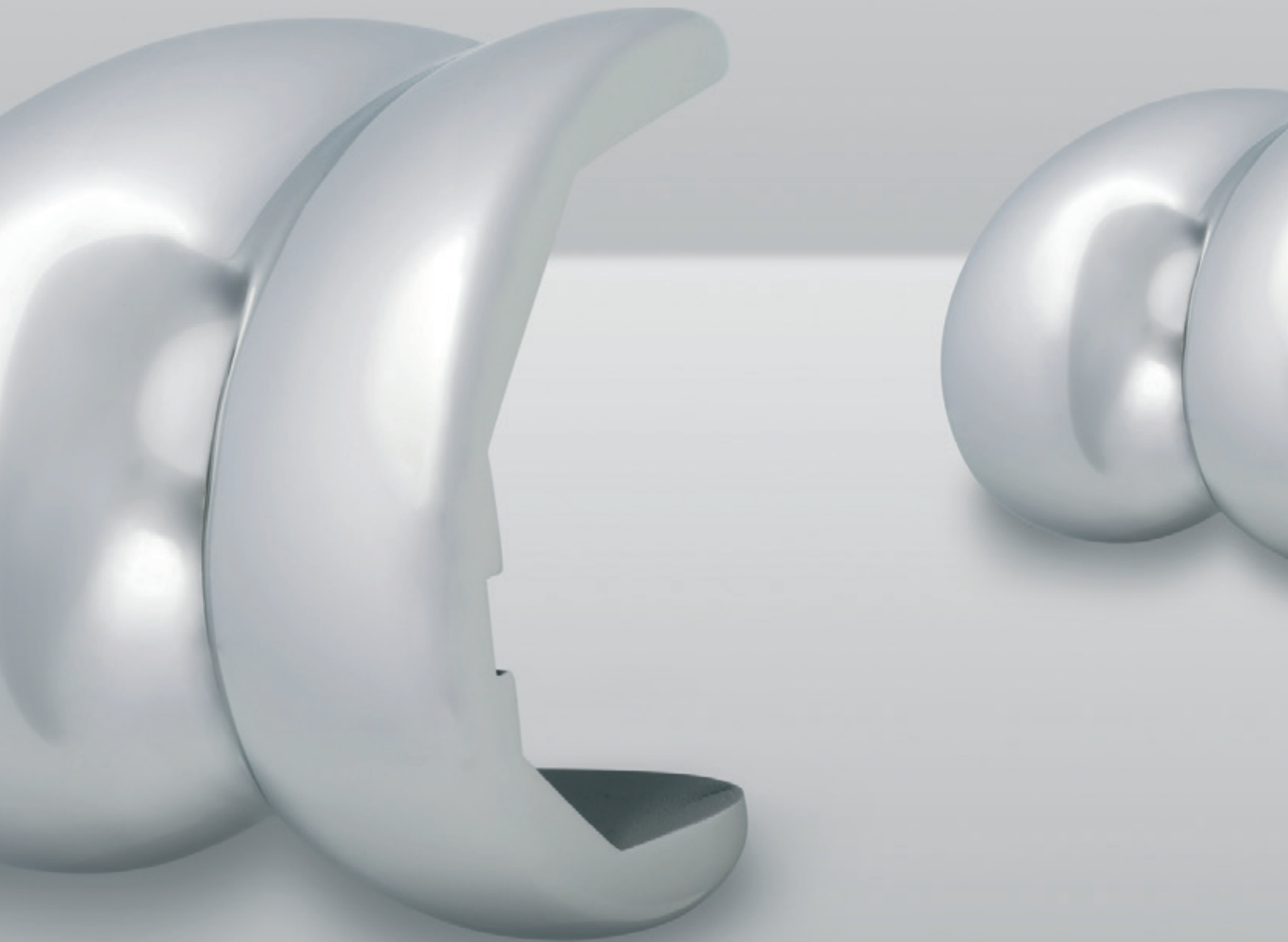
INTO EFFECT FROM: March 2011

@ SAMO SpA all rights reserved

Trekking® Primary Knee Integrated System

Rationale

The implant system

Trekking is one of the most advanced systems for total knee arthroplasty available today. All the principal technical and clinical issues have been analysed and solved in the most efficient way, with a lot of new and exciting solutions.

One of the most important features is the flexibility of all the Components - primary and revision, mobile and fixed, cruciate retaining, posterior stabilised or ultra-congruent, cemented and uncemented - which can be implanted with the same set of instruments, enabling the surgeon to choose the final Components at the very last moment, and even to combine “hybrid” primary and revision Components in a single implant for maximum customisation of the surgical treatment.

The Trekking System includes two sets of primary implants (Fig.1): cemented and uncemented, and a revision set.

All metal Components, except augments and stems are made of CoCrMo alloy, while all bearings are made of UHMWPE. Augments and stems are made of Ti6Al4V alloy.

Both Fixed and Mobile bearing systems, cemented and uncemented, come in cruciate retaining (CR) and posterior stabilized (PS) options; the revision set includes a semi-constrained fixed bearing, a non constrained, high flex fixed bearing, a single radius-high kinematic condyle, a Tibial plate, Tibial and Femoral stems, augments and offsets.

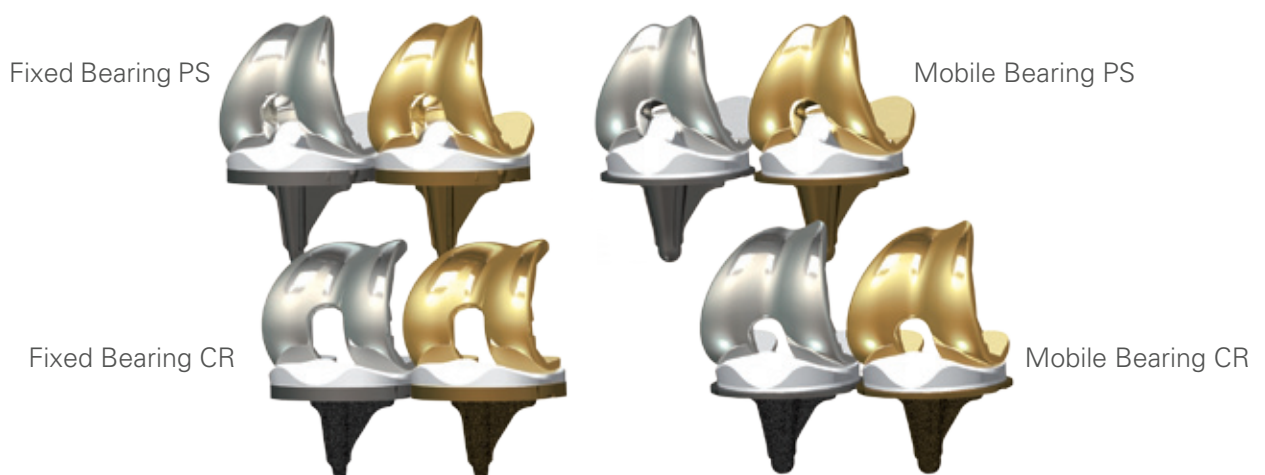
Fig.1 - Trekking Primary System

The press-fit version is available with a coating of pure titanium applied by vacuum plasma spray. The coated surface is perfectly flat and in contact with the bone. Both cemented and uncemented system are also available with a Nitrate Titanium (TIN) ipoallergenic coating

Fixed bearing

The fixed bearing system offers both Cruciate Retaining (CR) and Posterior Stabilized (PS) options. The connection between the insert and the Tibial plate is secured by means of a simple metal wire applied to the anterior part of the insert and a bowl-shaped Tibial plate with a mirror-polished 'floor'.

The Tibial plate is flat, with no additional spines, ripples or screw-holes, in order to minimize backside polyethylene wear.

The Tibial plate has a 4° posterior slope and a 6° articular surface slope.

*Fig. 2
Fixed Bearing
Components*

Mobile bearing

The mobile bearing system includes a Posterior Stabilized and an Ultra-Congruent Component. Thanks its 1:1 congruency, and to the anterior leap distance ranging from 6.6 (size 1) to 11.4 mm (size 5), the indication of the Ultra-Congruent implant is both for preserving and sacrificing the posterior cruciate ligament without compromising joint stability.

*Fig. 3
Mobile Bearing
Components*

The rotation, based on a central-pivot design, is obtained by a PE peg articulating into the Tibial plate keel hole. The insert peg has been designed with sufficient length to prevent dislocation and minimise polyethylene wear.

The Tibial plate has a 0° posterior slope and 6° articular surface slope.

Design features

During the last 30 years, knee implant design has had a dramatic evolution.

Huge problems have been overcome in many different fields, including anatomy, biology, mechanics and materials.

Today, implant design has achieved a satisfactory level of standardization.

However, some issues are yet to be completely addressed. The scientific challenge is now open in four main areas: *polyethylene wear, gait kinematics, bone preservation and specific surgical instruments.*

Polyethylene wear

The stress induced by the Femoral Component on the polyethylene bearing depends on three main factors:

- Weight of the patient
- Ligament tension
- Loading conditions (including the mechanical characteristics of the implant)

Surface roughness and stresses have a major impact on the wear process.

The best way to reduce stresses on the PE insert and to influence the loading conditions, is to maximize the contact area between the insert and the Femoral Component. In order to do this it is necessary to design the Femoral Component and the insert with the same radius, for perfect and complete contact (Figs.4-5).

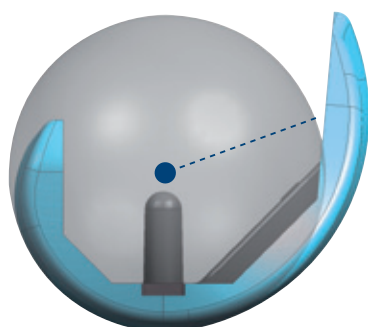
Fig. 4 - Sagittal single radius

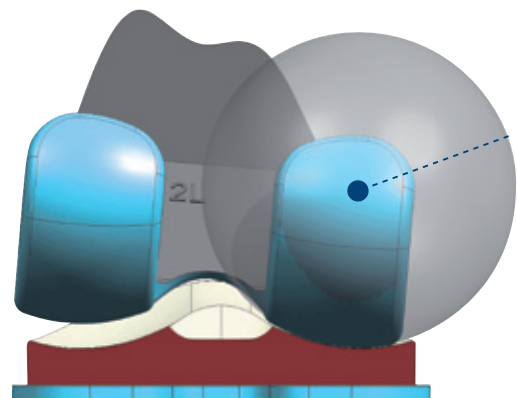
Fig. 5 - Frontal condylar single radius

The same concept applies to the central spine of the PS liner. In the vast majority of cases, the spine has a squared design, hit at each step by the posterior bar of the Femoral Component in a very narrow spot. The spine in the Trekking PS insert has a large surface and rounded corners in order to offer maximum and highly conforming contact to the congruent Femoral counterpart.

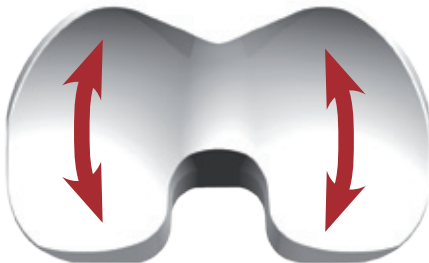
Fig. 6 - Spherical Track

Also the design of the articular surface in the fixed bearing knee has been subject to important improvements: the Spherical Track surface of the Trekking fixed bearing knee (Fig.6) enhances the contact area across the whole flexion and rotation range.

The polyethylene backside wear has been carefully addressed as well.

Fig. 7 - Bowl-shaped metal back

The simple mirror-polished, bowl-shaped metal back (Fig.7) and the connection mechanism with a metal wire engaging the notched edge of the metal back prevents polyethylene debris from spreading into the surrounding tissues.

Kinematics

In the Posterior Stabilized option, the shape of the insert is the most important factor affecting the relative movement of the femur and the tibia.

The geometry of the post in the Trekking PS insert has been studied to guide the movement of the femur throughout the gait cycle, to obtain a consistent and early rollback starting from 40° of flexion, with close-to physiological movement and enhanced ROM. The rollback feature also helps reducing the patellar pressure, which in return has a positive effect on the ROM.

The single radius for each Femoral Component in the frontal plane enables maximum contact in all the loading condition, even in a critical case of extreme lift-off one condyle is always completely in contact with the liner. This leads to higher stability and knee comfort.

The shape of the troclear groove is a good compromise between stability and patellar loading and optimizes the patellar tracking.

Bone preservation

Minimally Invasive Surgery is currently one of the most discussed techniques. A widely accepted aspects of MIS is the tissue sparing concept, a technique aiming at maximum tissue preservation and minimum damage, regardless of incision length.

The Trekking System has been carefully designed for maximum bone stock respect. The Tibial plate has been totally redesigned using a Finite Element Model, for exact evaluation of stresses and optimisation of thickness, which has been reduced from the usual 5 mm down to 3 mm still preserving safety.


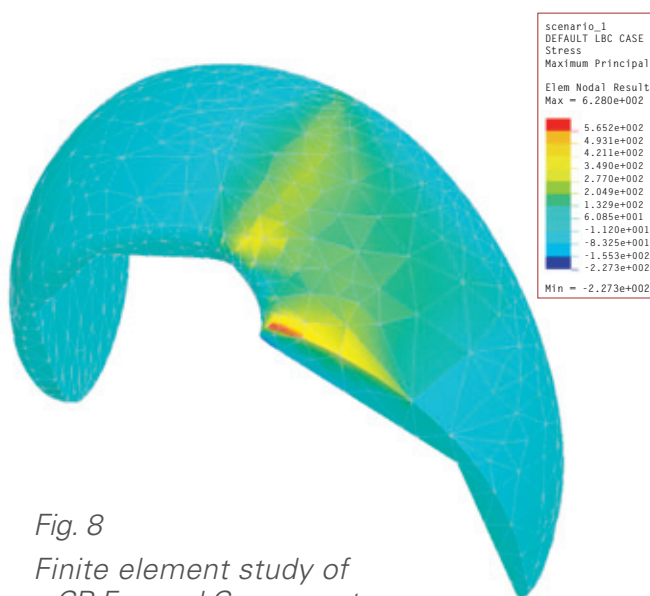
Fig. 8
Finite element study of a CR Femoral Component

The same process has been used with the Femoral Component. The FEM computer calculation allowed our designers to keep an optimal mechanical resistance still with a reduced thickness (8 mm) of the Femoral Component, both distally and posteriorly (Fig.8).

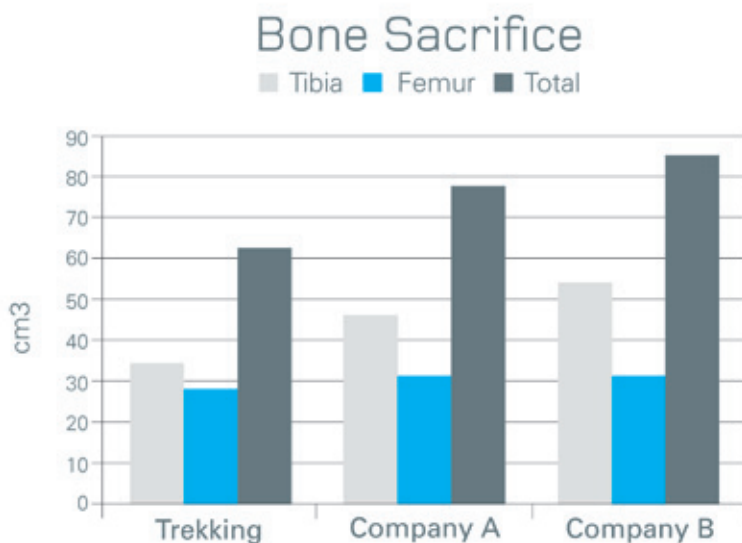Finally, thanks to the quality of the Ethylene Oxide-sterilised polyethylene, and the attention paid to tribological issues, the minimum Trekking insert thickness has been reduced down to 6 mm, still preserving the poly strength.

Thanks to this careful design, the Tracking System is one of the most bone preserving in the market.

Maximum total bone saving for a size 5 CR model is 20 cm³ (Fig.9) .

Fig. 9
Total bone sparing with a size 5 Trekking tibia, insert and femur vs two different competitor systems.

Innovative instruments

Unlike implants, the basic principles of TKA instruments have not changed for at least 25 years and concepts and techniques defined for the first systems have remained in use ever since.

With the Trekking instrument set, SAMO developed a really new, compact, modular, integrated instrumentation, making the procedure safer, more accurate, faster and simpler.

All the terms above are not casual. Let's analyse them:

- **Integrated:** all the Trekking Components (primary and revision) can be implanted by means of the same instrument set, with the addition of only 3 baskets for a complete revision implant.
- **Modular:** the instruments are available in containers that can be either used or left aside, depending on the implant system the surgeons is planning to use. As an example, Fig. 10 shows the solution for a PS, Fixed Bearing implant.

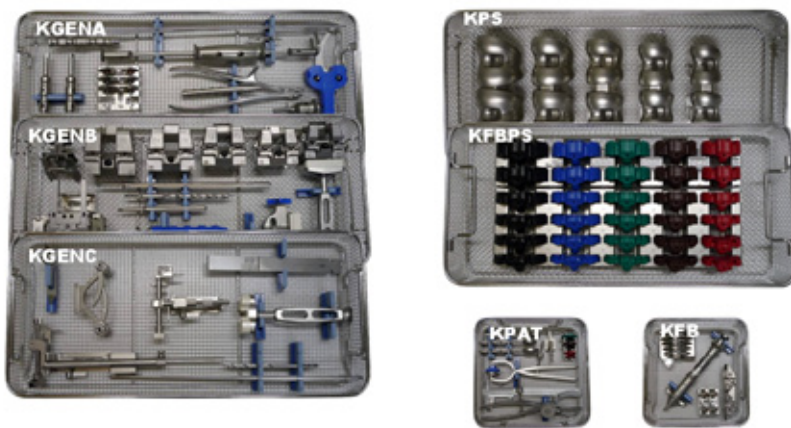
Fig. 10
Instruments set for a
Posterior Stabilized Fixed
bearing implant

- **Compact:** only 5 trays, plus two small square baskets, are necessary for one implant.
- **Simple and fast:** the unique *6-Actions Femoral Guide* (Fig.11) allows for time-saving, unique *3-Steps Femoral Cuts* (Fig.12).

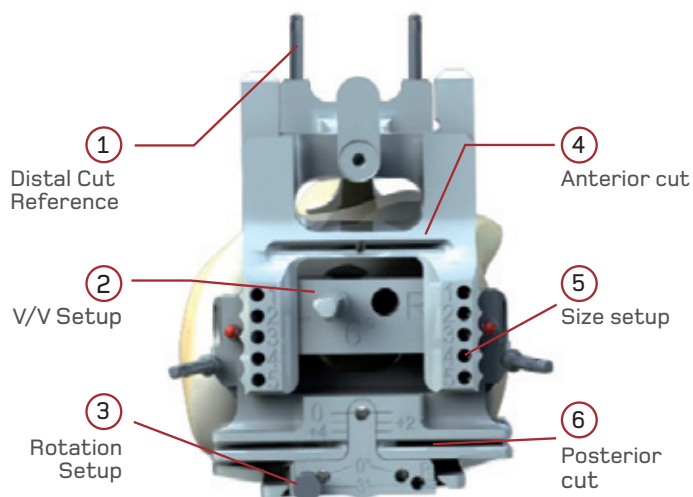
Fig. 11
6-actions Femoral guide

The classical bone-cut-sequence of most current instrument sets typically includes 5 steps: 1 – Varus Valgus setup; 2 – Distal cut; 3 – Size setup; 4 – A/P and oblique cuts; 5 – Box or trochlear groove cuts.

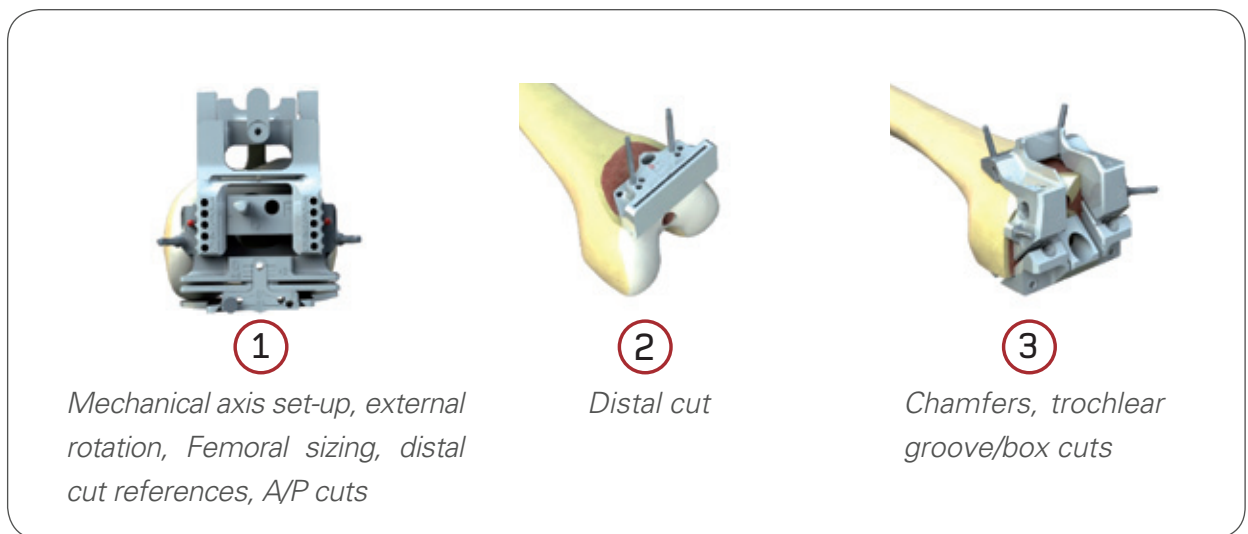
Fig. 12 - 3-Steps Femoral Cuts

• **Accurate:**

- Fixed, wide Femoral stylus with size reference for perfect sizing each time
- Stable guides - cuts are always precise and accurate allowing easy press-fit implant application.
- All the anatomical rotational landmarks (Whiteside line, Epycondilar line, posterior condyles) can be considered in order to determine the correct external rotation of the implant.
- Unique Reference Switch system for automatic anterior or posterior referencing, depending on the surgeon's preference.

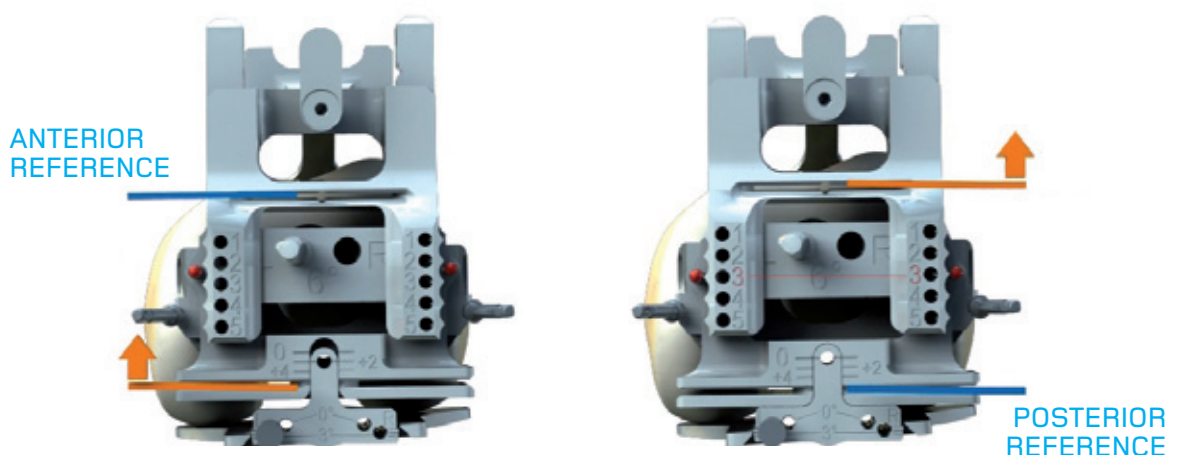
Fig. 13 - Automatic referencing Femoral guide

The system automatically switches between anterior and posterior reference adjusting modes, following the Femoral size choice of the surgeon (Fig.13). Therefore, for in-between sizes, if the

surgeon chooses the smaller size, the guide behaves as an anterior referencing system (the posterior reference, in such cases, would generate a risk of notching on the anterior cortex), while the guide behaves as a posterior referencing system if the surgeon chooses the larger size (an anterior reference system, reducing the flexion gap, would cause tightness in flexion). In current systems, either the choice is forced (always larger OR always smaller size every time), or the surgeon must switch instruments in order to change the reference from posterior to anterior or vice-versa.

- **Safe:**

- *Recutting is allowed with minimal time loss and without affecting cut precision -* Thanks to the particular sequence of the cuts, tissue balance becomes a central step in the technique, and recutting, when needed, it is a simple and fast procedure.
- *100% notching avoided -* Thanks to the floating anterior cut guide, notching is made impossible. Regardless of how precisely the cutting guide is placed, the cut will always be over the surface of the anterior cortical bone (Fig.14).

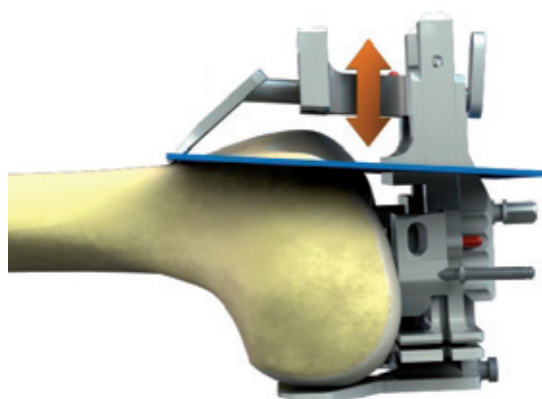
Fig. 14 - 0-notching system

These are only general concepts about **the highly innovative Trekking System Instrument Set**. Please, read the following chapters to learn more about all the smart solutions introduced by this system.

Trekking® Primary Knee Integrated System

Surgical Technique

The surgical technique of the TREKKING® Primary Knee System has been optimized in terms of precision, balance check, accuracy and execution speed.

The surgery can be schematized in the following 3 phases:

1. **Main cuts**
2. **Balance check**
3. **Finishing**

Performing tissues balance checks before chamfering allows for precise and quick recuts, when needed. When balancing is correct, the femur can be chamfered. The surgeon can then perform the trial reductions; any correction performed at this stage does not compromise the bone-implant fit.

This surgical sequence has been designed to obtain maximum precision and easy solution of any unsatisfactory situations.

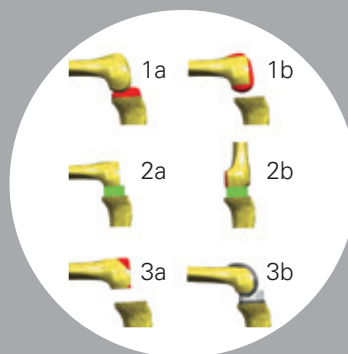
While it is possible to start either from the tibia or from the femur, the Tibial resection gives the surgeon a good benchmark to perform the Femoral cuts, and more room to operate in the Femoral part. Leaving the chamfers as the last cuts allows a much more accurate measurement of the articular gap and recutting is easier, if any is needed. The functional augmentation system of the spacer helps the surgeon select the correct thickness of the insert.

At any stage, the surgeon can check articular alignment accuracy with a metal rod inserted in the eyelets available in most instrumentation Components.

1. Main cuts

2. Balance check

3. Finishing

*Conceptual phases of the TREKKING®
Mobile Hollow surgical technique*

ADVICE: The instrument set requires 1.27 mm (1/20") thick, 13 mm (1/2") wide and 90 mm (6.5") long blades. Different blade parameters may hinder cutting operations. In particular, thicker blades are not compliant with the height of the guides, and must be avoided.

LEGEND:

[.] - reference to a device of the instrument set, see the section at the end of the document

() - reference to a detail of the image

■ - specific Surgical Technique

■ - specific Surgical Technique


■ - specific Surgical Technique

■ - general Surgical Technique

USE OF THE PINS		
45. Self-drilling Pin. GS.C0800		Fixation with parallel pins
46. Headed Medium Pin GS.C0600		Sizing
47. Headed Short Pin GS.C0700		Parameters set-up, trial Tibial plate fixation
48. Self-drilling Pin w. abutment GS.C0810		Fixation with slanting pins

1.

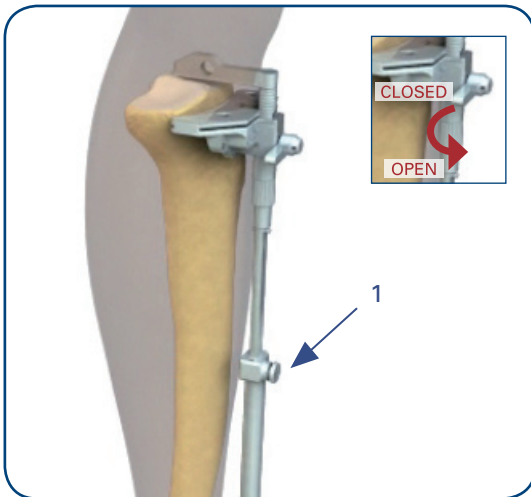MOBILE BEARING

The proposed surgical technique is based on a “tibia first” approach. For a ‘femur first’ technique, steps 1 to 7 should be read after step 15.

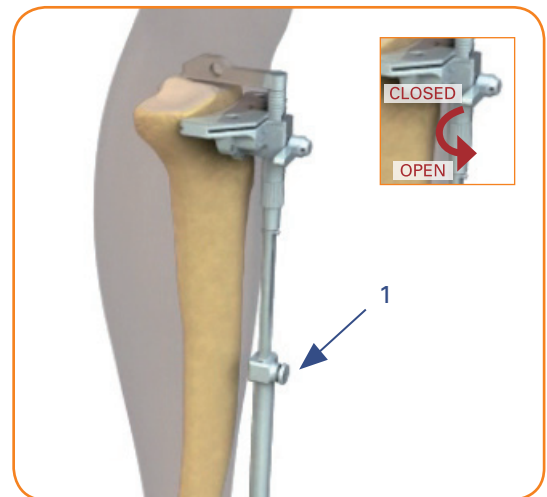
The Tibial System is to be mounted by inserting the Extramedullary Tibial Guide [13.] into the Extramedullary Tibial Guiding Tube [14.] and then lifting the Ankle Clamp [15.] into the guide in the lower part of the Guiding Tube. Place the MBH Tibial Cut Block [11. MBH] in its lodging at the top of the Extramedullary Tibial Guide, lock the Cut Block by pushing the lever and then turn it 90°.

1.


FIXED BEARING & TCK

The proposed surgical technique is based on a “tibia first” approach. For a ‘femur first’ technique, steps 1 to 7 should be read after step 15.

The Tibial System is to be mounted by inserting the Extramedullary Tibial Guide [13.] into the Extramedullary Tibial Guiding Tube [14.] and then lifting the Ankle Clamp [15.] into the guide in the lower part of the Guiding Tube. Place the FB Tibial Cut Block [11. FB] in its lodging at the top of the Extramedullary Tibial Guide, lock the Cut Block by pushing the lever and then turn it 90°.

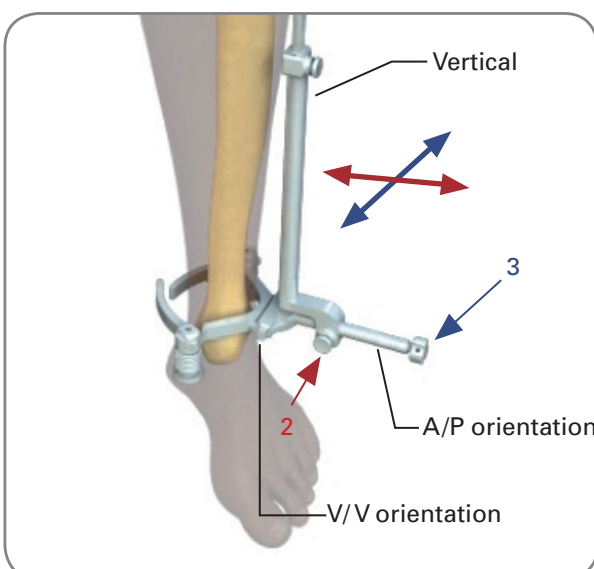2.

Comfortable push-buttons (1,2) and a knob (3) allow the orientation of the cut in the proximal-distal, medio-lateral and Antero-Posterior (A/P) directions. For better stability, a short headed pin may be inserted into the hole placed in the upper part of the fixation arm (2 in step 3).

Regardless of whether you are using a mobile or fixed bearing implant, the alignment guide and anatomical axis of the tibia must be parallel. The different slope of the cut (0° for the mobile bearing implant and 4° for the fixed bearing implant) is given by the different cutting block.

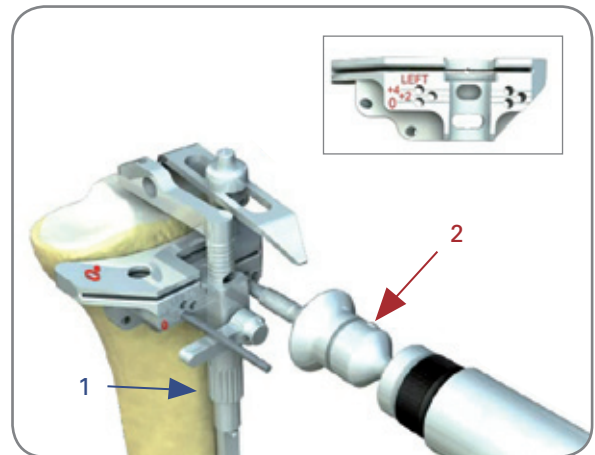
NOTE: Each sign marked on the A/P orientation guide counts as a 0.5° of slope of the Tibial cutting block; the same applies to V/V inclination, i.e. by moving the vertical bar one space in A/P or V/V direction, the A/P or V/V inclination of the Tibial cutting block changes by ±0.5°.

3.


The proper height of the Tibial cut is set by means of the two-tips Tibial Stylus [12.], which has to be placed on the inferior point of the healthiest part of the Tibial plate. One tip of the stylus is marked as "Slot" and must be used if the surgeon intends to cut using the slot of the cutting block as a guide; the other one is marked as "Top", and must be used if the cut is performed leaning the blade onto the flat top of the guide.

In both cases, the cut performed is 9 mm, i.e. the minimum total thickness of the Tibial implant (insert + metal back). A threaded knob (1) may be used to lower or raise the cut.

When the Tibial Cutting Block is in the correct position, pins must be placed in the "zero" holes. The "+2" and "+4" holes are for recutting only. The universal Drill Adapter (2) [34.] may be used. The setup of the Tibial guide can be checked by means of the Resection Tester [28.].

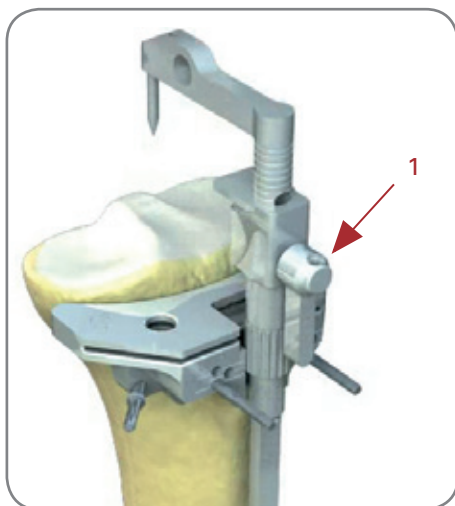4.

The Tibial Cutting Block can be completely freed just by turning the locking lever (1) placed in front of the block and by using the "fork tip" of the Extractor [41.].

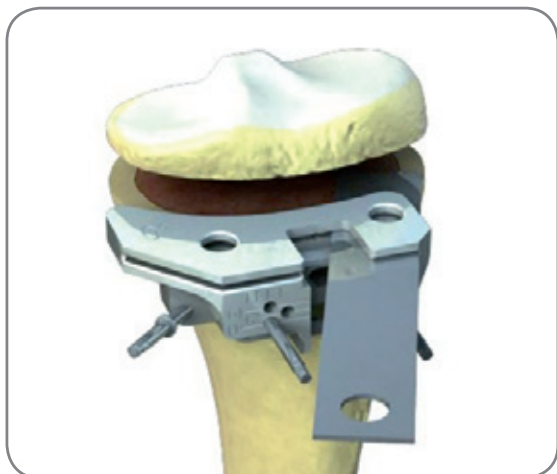
For good block stability, it is advisable to spike the cutting block with three pins as shown in steps 5 and 6.

5.

Before cutting, the alignment with the Tibial axis can be checked by inserting the checking rod [24.;25.] into the hole of the modular handle [33.], previously fixed on the Tibial Cutting Block [11.].

6.


As previously stated, the cut can be made either using the slot or the top plane as guide, depending on the choice made in step 3.


Remove the cutting guide leaving the parallel pins in place for possible recuts whether the slot or top end of the stylus has been used.

7.

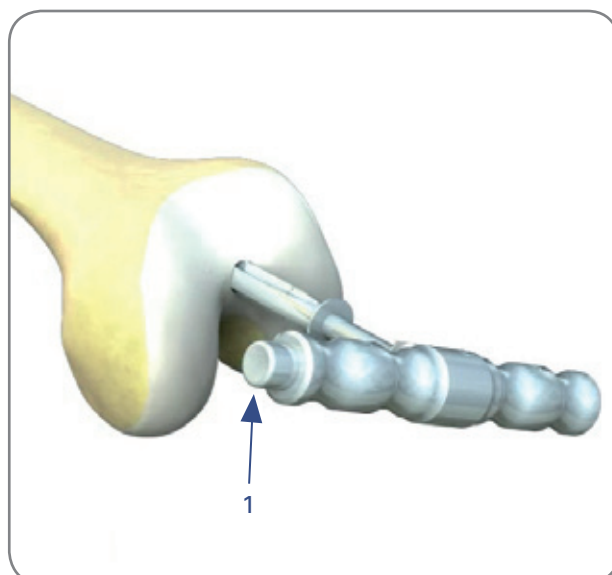After distal femur drilling has been performed, the Femoral Intramedullary Rod [2.], provided with a modular handle [23.] is placed through the hole. Make sure the Rod is fully inserted. The handle can be placed and removed with a push-button (1).

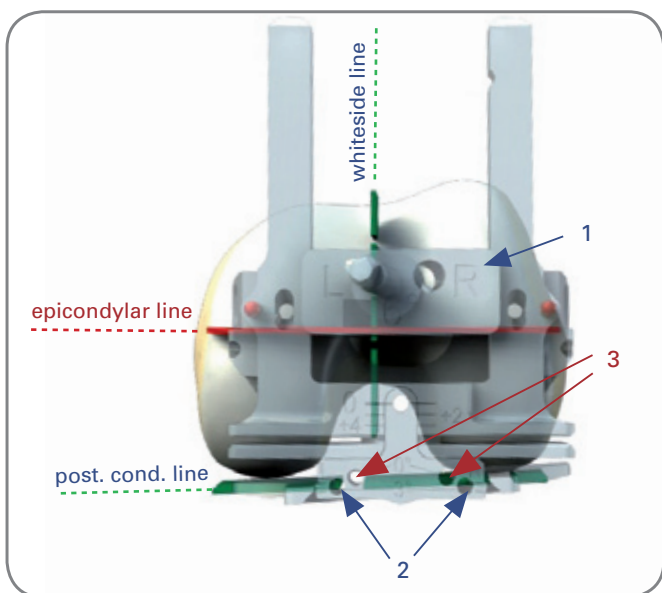8.

Before placing the Orientation Jig [4.b.], choose the proper varus-valgus and interanal-external rotation angle.

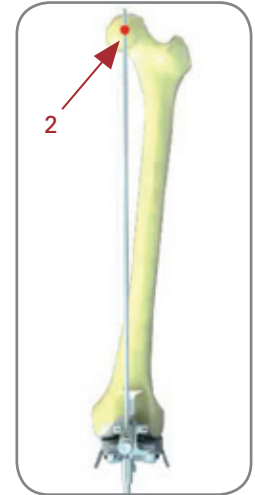
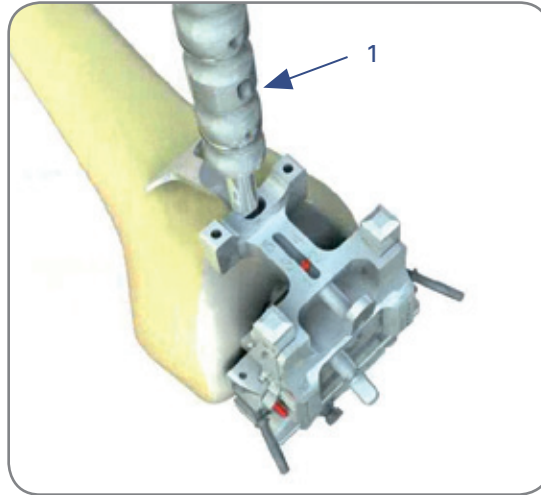
The varus-valgus angle is set using the varus-valgus block (1) [5.], while the intra/extra rotation of the implant is set either by introducing a short headed pin [47.] in the 0° (2) or 3° (3) hole or by aiming at the Whiteside line through the vertical bars or by aligning the guide with the intra-epicondylar axis.

In all cases, both posterior prongs must be kept in contact with the posterior condyles.


9.

By means of the checking rod inserted into the hole of the modular handle [33.], (1), it is possible to check the conformity between varus-valgus block and individual patient's anatomy.

If the varus-valgus angle is placed correctly, the checking rod will ideally cross the Femoral rotation centre (2).

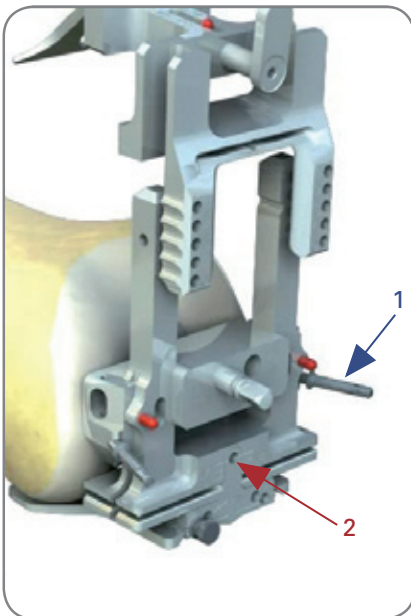10.

Fix the orientation jig by means of [48.] pins (1). Once the jig has been properly placed and fixed, let the Size Jig [4.a.] slide into the Orientation Jig, until the stylus touches the anterior part of the femur.

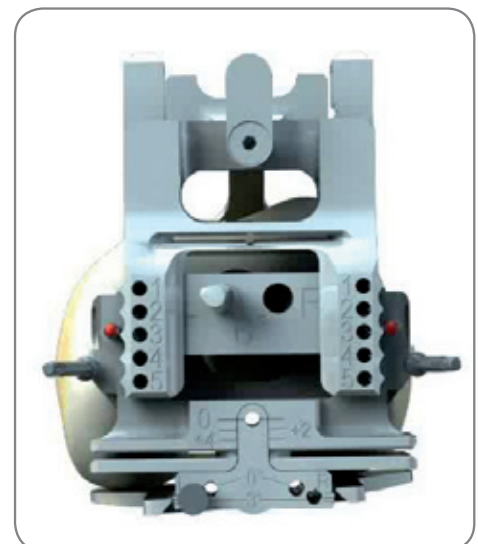
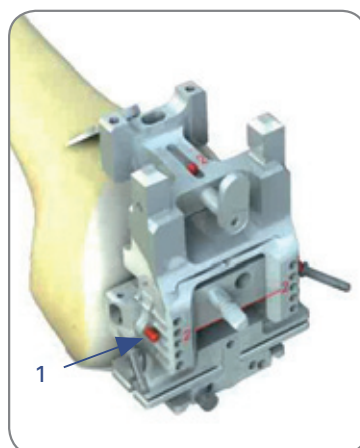
Observing the image below, note that both the anterior stylus and the posterior prongs are touching the bone. If necessary remove interposing soft tissues and osteofites. Set the stylus at default temporary size '3' on the scale on the upper part of the guide (step 11, left).

ADVICE: Until step 12, make sure that the graduated scale (2) is set at zero by placing a pin on the hole (2), which must be removed before inserting the pin into the size hole, as indicated in step 12.


11.

Read the size at the level of the red pins (1), flush with the centre of the holes. Often the measurement does not match with a precise size. In these cases, it is advisable to prefer the larger size only when the pointer is very close ($\frac{2}{3}$ or more) to the upper size. Set the chosen size on the stylus size scale (below).

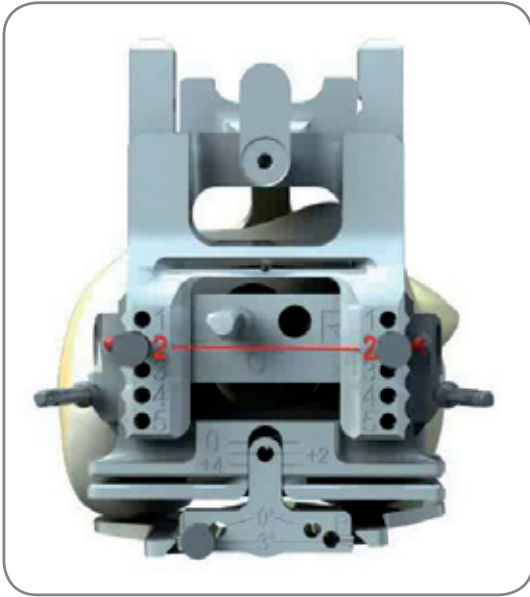12.

IMPORTANT NOTE: If the smaller size has been chosen, remove the optional pin in central hole now.

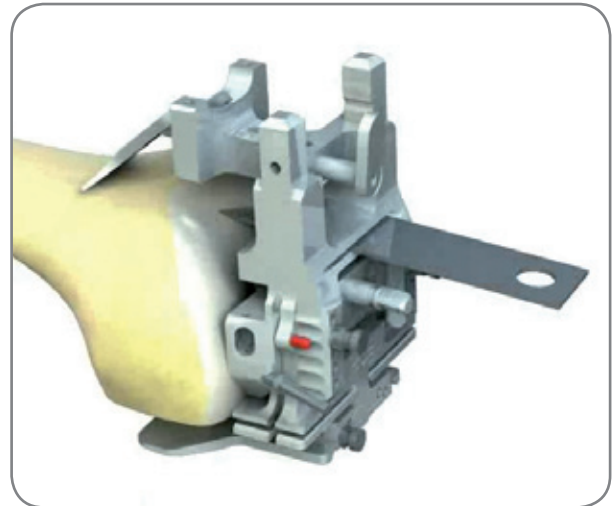
Introduce two headed pins [46.] into the hole corresponding to the chosen size. The device will automatically slide, setting the cutting slots to the chosen size.

ADVICE: In order for the device to reach the best position, the pins must NOT be hammered, but pushed by hand instead, vibrating it a bit in the vertical direction to facilitate the sliding of the mechanisms. As the system has slid into the correct position, and the pins have reached the Femoral bone, use a hammer to fix the pins to the bone.


If the smaller size is chosen, the graduated scale (1) will indicate +2 or +4; these numbers can be used in the distal cut (step 15).

13.

Make the anterior and posterior cuts through the corresponding slots. The stylus always indicates the proximal border of the anterior cut.

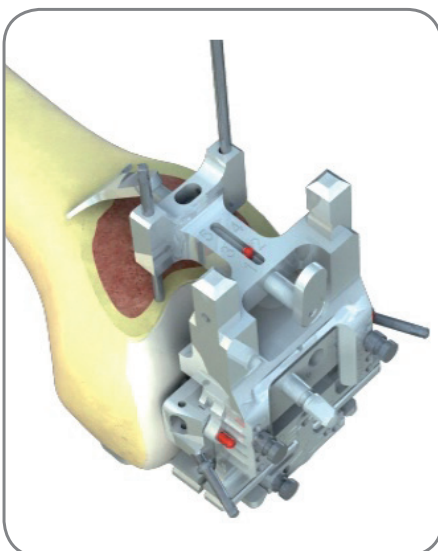14.


Insert two pins [45.] into the anterior holes, then remove all devices, leaving the 2 anterior pins in place. To remove the devices, remove all the Headed Pins, then lift and remove the Size Jig [4a.], then extract the Orientation Jig [4b.] and the Intramedullary Rod.

15.


The Distal Femoral Cutting Block [7.] presents 3 couples of holes, marked as 0, +2, +4. In order to balance ligament tension in flexion and extension, it is possible to use the cut depth following the number indicated in the lower graduated scale (1) in step 12.

ADVICE: *The procedure described above is optional. In any case, the final evaluation of the cut depth must be made by the surgeon, following the actual anatomy of the patient.*

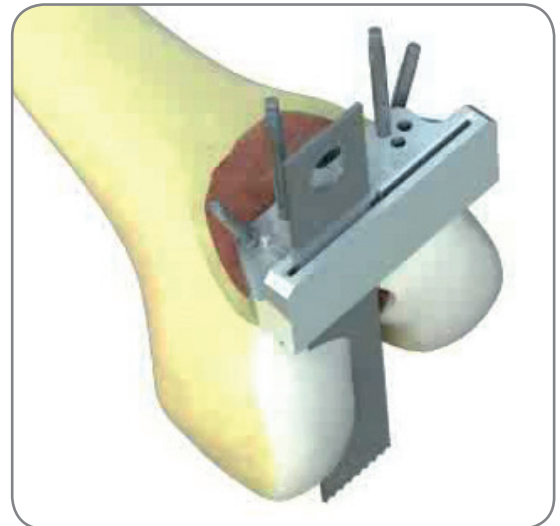Cut the distal condyles by using the slot and by shielding the tibia with the Hohmann retractor [50.], then remove the Distal Cut Block, leaving the parallel pins in place.

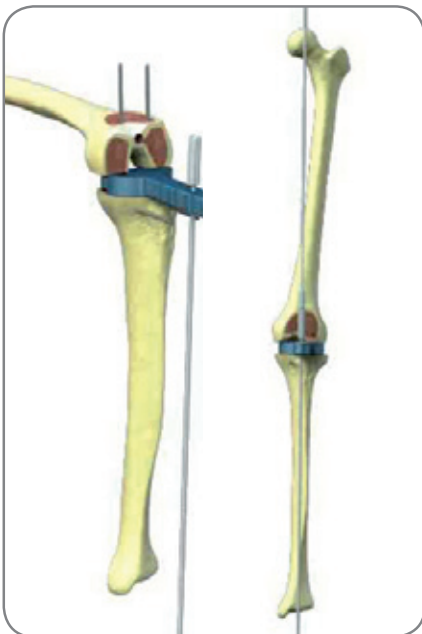16.

At this stage, it is very convenient to perform the balance checks by means of the appropriate spacer [26.], in order to check the Femoral and Tibial Component matching.

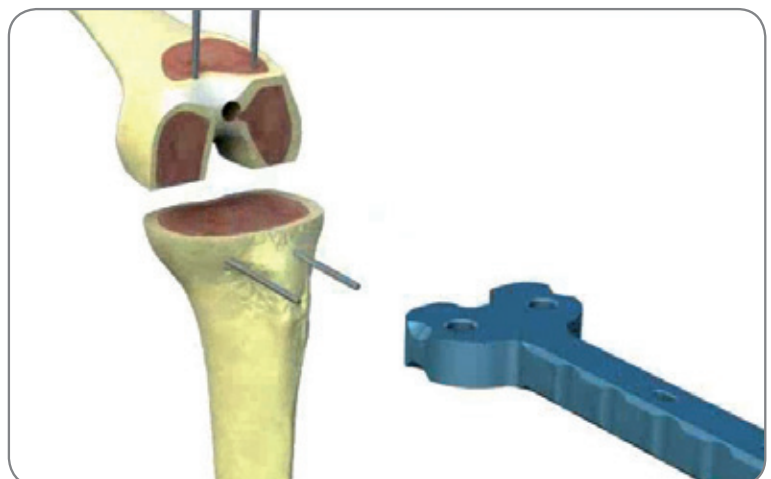
It is important to do it before chamfering, to obtain a more precise measurement and a much easier recutting, if required.

A very easy recutting is enabled by the presence of the pins in the original position (steps 6; 15).


Screwing the Male Control Rod [25.] onto the Female Control Rod [24.], check the mechanical axis alignment.

17.


Spacer Augmentations [35.] are marked with thickness size (thickness reference number) of the corresponding insert. Read the thickness reference number on the used augmentation (+2, +4, +7), so as to be aware of the insert dimension. If the tension of the collateral ligaments is satisfactory without using any augmentation, then use the +0 insert.

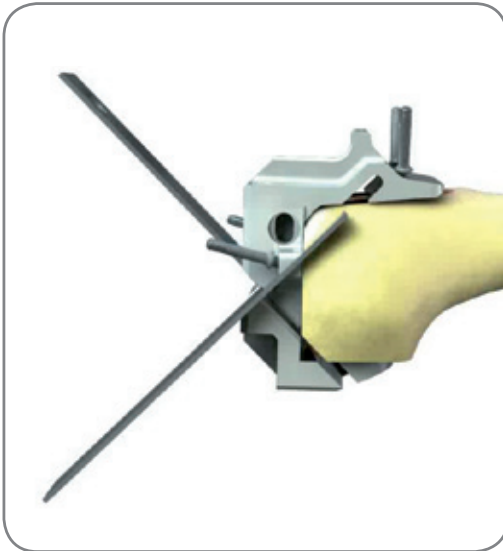18.

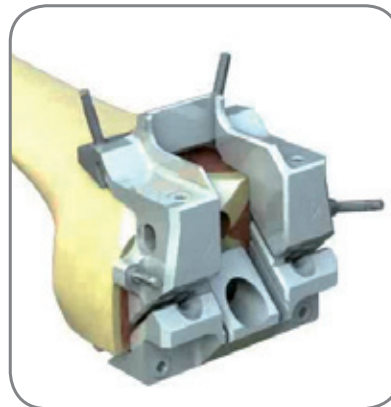Except for the pin slots, the Femoral Chamfering Jig [19.] has the same lateral dimension as the Femoral Component. This helps properly place the jig, and have a preview of the implant final setup.




Make the oblique anterior and posterior cuts using the appropriate guides.


19.

POSTERIOR STABILIZED & TCK

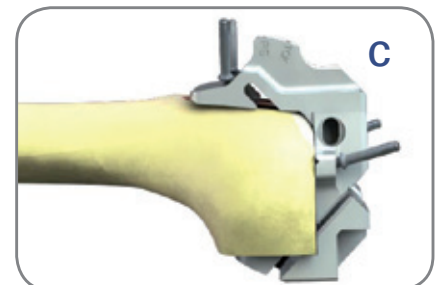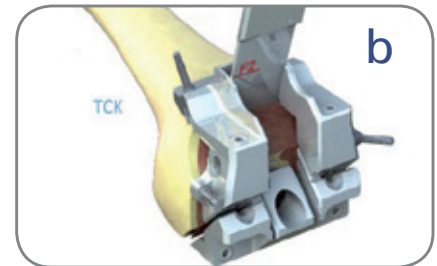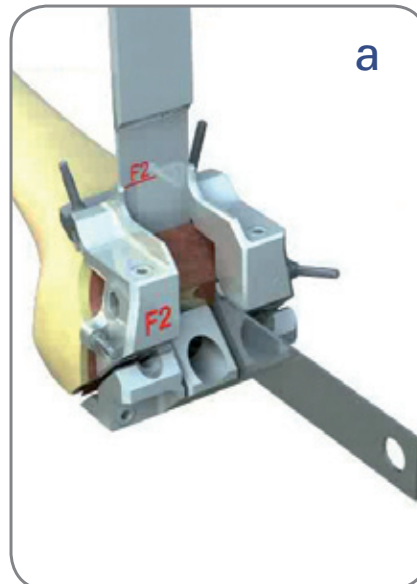In PS and TCK implants, after step 18, make the proximal box cut (1) using the osteotome [38.] and the appropriate guiding rail (figs a, b, c); then complete the box by leaning the sawblade onto the two lateral and medial planes (2), keeping the osteotome as a shield. Then remove the osteotome and perform the cut shown in step 20; then remove the jig. Do not drill the holes for the pins, which are not included in the PS implant.

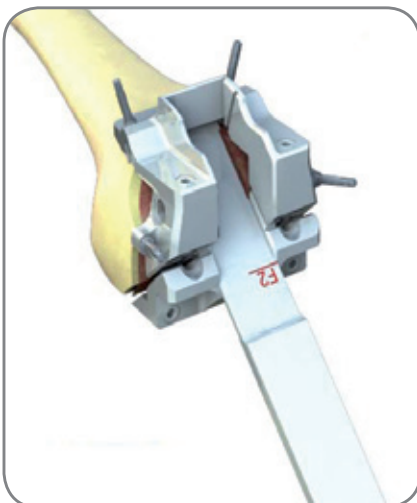
20.


For both CR and PS implant, make the central oblique cut leaning the blade on the oblique plane, or using the osteotome [38.] through the appropriate guide. In CR implants, drill the holes for the pin (drill [6.]) and remove the Jig.

21.

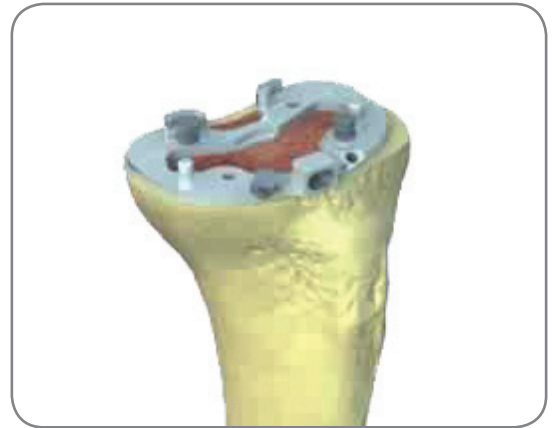Put a Trial Tibial Tray [30.] in place after engaging the modular handle [33.]. Change the tray size until you find the right one for the Tibial Component. Spike it with at least four headed pins.

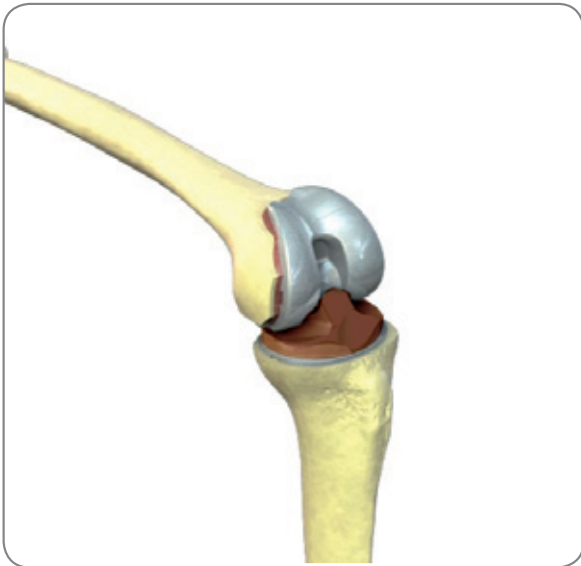22.

Put the Trial-Insert Adapter [8.] into its slot on the Trial Tibial Tray. Follow the indications on thickness (step 17) and choose the appropriate trial insert [53.; 54.; 55.; 56.] taking into account that, with the Mobile Hollow design, the insert must match the size of the Femoral Component exactly, while the Tibial Component can be ± 1 size.

However, in case of FB, the insert must match the size of the Tibial Component exactly, while the Femoral Component can be ± 1 size.

Place the Trial Femoral Component [42.; 51.] and perform flexion and extension trial reduction to check the fit of the whole implant.

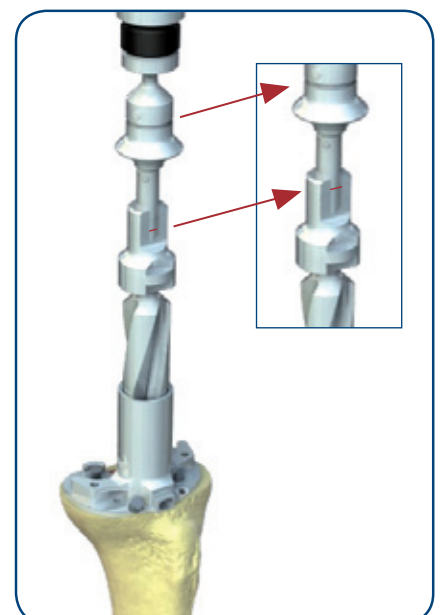
23.

MOBILE BEARING

Remove all the trial Components, except for the Trial Tibial Tray. In order to extract the Trial Femoral Component, insert the flat tip of the Extractor [41.] into the tracks placed on either side of the trochlear groove of the trial Component.

Set the drill height in the MBH Tibial Drill [31.] at the size chosen in step 23.



Place the MBH Tibial Drill Guide [21.] into its slot on the Trial Tibial Tray. Use the MBH Drill Guide [31.] to drill the hole.

In the case of TCK, after removing all the trial components except for the Trial Tibial Tray, place the TCK Tibial Drill Guide [18.] into its slot on the Trial Tibial Tray. Use the TCK Drill Guide [17.] to drill the hole.

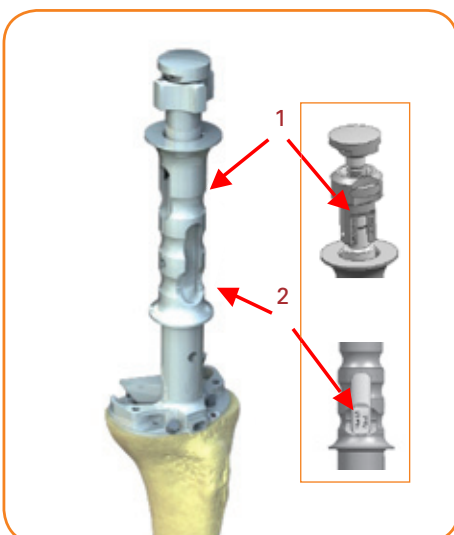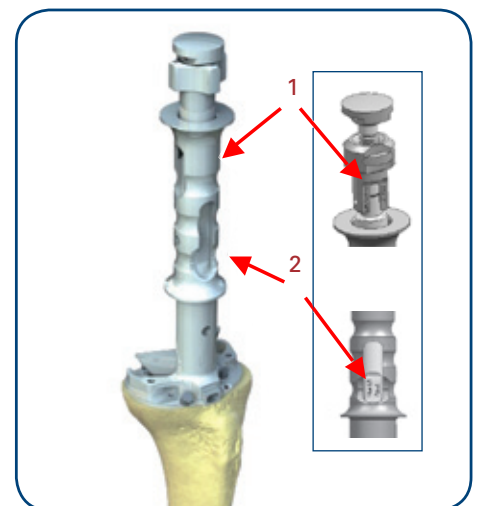Remove all the instruments, except for the Trial Tibial Tray. Set the Tibial Plate size on the upper graduate scale of the harrow broach (1). Place the Arrow Broach (0°) [22.] into its slot on the Trial Tibial Tray. Hammer in and then check full penetration of the broach using the lower graduate scale (2).

Remove all the instruments, except for the Trial Tibial Tray. Set the Tibial Plate size on the upper graduate scale of the harrow broach (1).

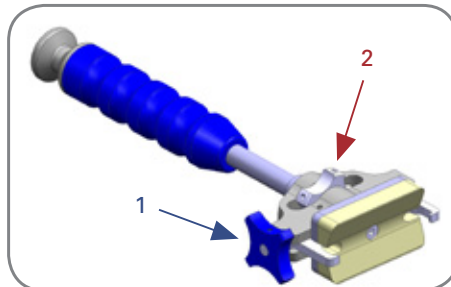
Place the Arrow Broach (4°) [22.] into its slot on the Trial Tibial Tray. Hammer in and then check full penetration of the broach using the lower graduate scale (2).

25.


Fix the final Femoral/Tibial Component on the Positioner [43.] and hammer in the Component, then reduce the joint. Place the plastic liner on top of Tibial plate. In the case of Fixed Bearing follow this procedure:

- push the liner posteriorly until it's perfectly flush and engaged with the posterior lip of the Tibial Component
- push with the finger or by means of the Tibial impactor snapping the liner into the Tibial plate

ADVICE: Engage the Component by turning the lateral screw (1), push the plastic piston against the Component by turning the central knob (2).

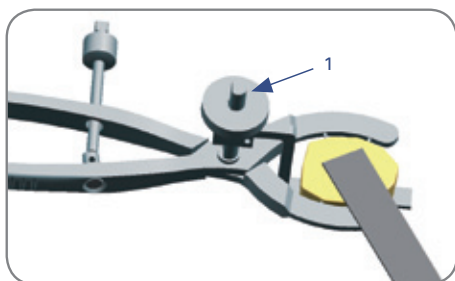26.

When the Patellar Component is to be implanted, the first step is measuring the thickness of the patella. Subtract 9 mm (the thickness of the patellar Component) from the measured value, then set the resulting value on the graduated scale carved on the hinge pole (1) of patellar resection pliers [37.].

Grasp the patella using the Patellar Pliers and lean it on the stylus. Pay attention to patellar resection slope. Cut the posterior part of the patella.

27.

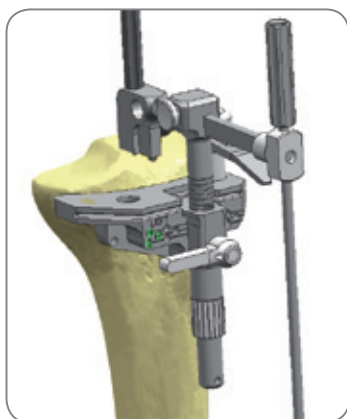
Among available patellar Components, choose the one that best fits the resection.

Holding the Component by the side, make three holes with the Patellar Pin Drill.

Trial patellar Components are available in the instrument set. During the cementing process, the Patellar Cementing Pliers [39.] can be used to keep the patellar Component pressed onto the bone.

Intramedullary alignment

The figure to the right shows the intramedullary system with the Checking Rod [24.] mounted. Open the path for the intramedullary Tibial rod using the Tibial twisted drill [9.]. Insert the intramedullary Tibial rod [10.] and mount the intramedullary Tibial guide (1) [29.]. Both in the extra and intramedullary technique, it is advisable to insert the slanting pin after removing the Tibial guide, in order to find the right distance from the block to the tibia.


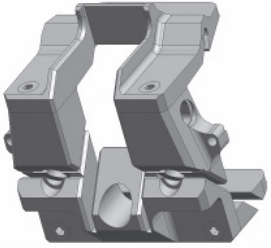
The use of the endomedullary guide with heavily bent tibias is not advised.

Trekking® Instrument Set

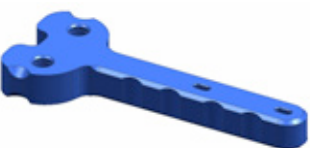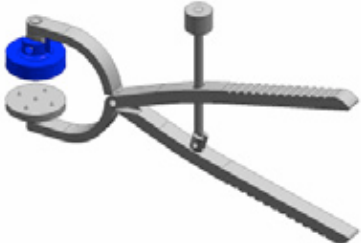
Images are not full scale; not all the parts are represented.

1. Femoral Twisted Drill GS.F0100		
2. Femoral Intramedullary Rod		L100 mm GS.F0200
		L240 mm GS.F02002
3. Femoral Alignment System GS.F0300		
4. Size Jig (4.a), Orientation Jig (4.b)	 4.a	
	 4.b	
5. Varus-valgus Block		Angle 4° GS.F0304
		Angle 6° GS.F0306
		Angle 8° GS.F0308
6. Twisted Drill for Femoral pins GS.F1400		
7. Distal Femoral Cutting Block GS.S2000		
8. MBHTrial-Insert Adapter GS.T1500		


9. Tibial Twisted Drill GS.T0100			
10. Tibial Intramedullary Rod GS.T0200			
11. Tibial Cutting Block		MBH (0°)	RIGHT GS.T9000
			LEFT GS.T8000
		FB (4°)	RIGHT GS.T8100
			LEFT GS.T9200
12. Tibial Stylus GS.T0400			
13. Extramedullary Tibial Guide GS.T0700			
14. Extramedullary Tibial Guiding Tube GS.T0300			
15. Ankle Clamp GS.T0600			
16. Tapered Adapter GS.T0600			
17. TCK Tibial Drill GS.R0014			
18. TCK Tibial Drill Guide GS.R0011			


19. Femoral Chamfering Jig		Size 1 GS.F1200CK
		Size 2 GS.F1100CK
		Size 3 GS.F1000CK
		Size 4 GS.F0900CK
		Size 5 GS.F0800CK
20. Femoral Impactor GS.F1500		
21. MBH Tibial Drill Guide GS.T2600		
22. Arrow Broach		MBH (0°) GS.F7300
		FB (4°) GS.T7400
23. Endomedullary Rod Handle GS.C0100		
24. Female Checking Rod GS.C0400		
25. Male Checking Rod GS.C0404		
26. Spacer GS.T9700		
27. Resection Tester GS.C0300		
28. Patellar Jig		GS.P1100
		GS.P1200
		GS.P1300

29. Intramedullary Tibial Guide GS.T0500			
30. Tibial Tray		MBH	Size 1 GS.T1000
			Size 2 GS.T1100
			Size 3 GS.T1200
			Size 4 GS.T1300
			Size 5 GS.T1400
		LB	Size 1 GS.T1001
			Size 2 GS.T1101
			Size 3 GS.T1201
			Size 4 GS.T1301
			Size 5 GS.T1401
31. MBH Tibial Drill GS.T2700			
32. Tibial Impactor GS.T3100			
33. Fast Handle GS.C0200			
34. Drill Adapter GS.T3000			
35. Spacer Augmentations		+2	GS.T9300
		+4	GS.T9400
		+7	GS.T9500
		+10 (FB)	GS.T9640
		+13 (FB)	GS.T9680
36. Patellar Pin Drill GS.P0300			
37. Patellar Resection Pliers GS.F1700			

38. Osteotome GS.F2100																								
39. Patellar Cementation Pliers GS.1600																								
40. Pin Impactor GS.C0500																								
41. Extractor GS.C1000																								
42. CR Trial Femoral Comp. R/L		<table border="1"> <tr><td rowspan="5">DX</td><td>Size 1</td><td>GS.F3000</td></tr> <tr><td>Size 2</td><td>GS.F3010</td></tr> <tr><td>Size 3</td><td>GS.F3020</td></tr> <tr><td>Size 4</td><td>GS.F3030</td></tr> <tr><td>Size 5</td><td>GS.F3040</td></tr> <tr><td rowspan="5">GS</td><td>Size 1</td><td>GS.F3050</td></tr> <tr><td>Size 2</td><td>GS.F3060</td></tr> <tr><td>Size 3</td><td>GS.F3070</td></tr> <tr><td>Size 4</td><td>GS.F3080</td></tr> <tr><td>Size 5</td><td>GS.F3090</td></tr> </table>	DX	Size 1	GS.F3000	Size 2	GS.F3010	Size 3	GS.F3020	Size 4	GS.F3030	Size 5	GS.F3040	GS	Size 1	GS.F3050	Size 2	GS.F3060	Size 3	GS.F3070	Size 4	GS.F3080	Size 5	GS.F3090
DX	Size 1	GS.F3000																						
	Size 2	GS.F3010																						
	Size 3	GS.F3020																						
	Size 4	GS.F3030																						
	Size 5	GS.F3040																						
GS	Size 1	GS.F3050																						
	Size 2	GS.F3060																						
	Size 3	GS.F3070																						
	Size 4	GS.F3080																						
	Size 5	GS.F3090																						
43. Femoral Component Positioner GS.0300																								

44. Self-drilling pin Driver GS.C0900																								
45. Self-drilling Pin. GS.C0800																								
46. Headed Medium Pin GS.C0600																								
47. Headed Short Pin GS.C0700																								
48. Self-drilling Pin w. abutment GS.C0810																								
49. Trial Patellar Component		<table border="1"> <tr><td>Size 1</td><td>GS.P0800</td></tr> <tr><td>Size 2</td><td>GS.P0900</td></tr> <tr><td>Size 3</td><td>GS.P1000</td></tr> </table>	Size 1	GS.P0800	Size 2	GS.P0900	Size 3	GS.P1000																
Size 1	GS.P0800																							
Size 2	GS.P0900																							
Size 3	GS.P1000																							
50. Hohmann's Lever GS.C1620																								
51. PS Trial Femoral Comp. R/L		<table border="1"> <tr><td rowspan="5">DX</td><td>Size 1</td><td>GS.F3100</td></tr> <tr><td>Size 2</td><td>GS.F3110</td></tr> <tr><td>Size 3</td><td>GS.F3120</td></tr> <tr><td>Size 4</td><td>GS.F3130</td></tr> <tr><td>Size 5</td><td>GS.F3140</td></tr> <tr><td rowspan="5">GS</td><td>Size 1</td><td>GS.F3150</td></tr> <tr><td>Size 2</td><td>GS.F3160</td></tr> <tr><td>Size 3</td><td>GS.F3170</td></tr> <tr><td>Size 4</td><td>GS.F3180</td></tr> <tr><td>Size 5</td><td>GS.F3190</td></tr> </table>	DX	Size 1	GS.F3100	Size 2	GS.F3110	Size 3	GS.F3120	Size 4	GS.F3130	Size 5	GS.F3140	GS	Size 1	GS.F3150	Size 2	GS.F3160	Size 3	GS.F3170	Size 4	GS.F3180	Size 5	GS.F3190
DX	Size 1	GS.F3100																						
	Size 2	GS.F3110																						
	Size 3	GS.F3120																						
	Size 4	GS.F3130																						
	Size 5	GS.F3140																						
GS	Size 1	GS.F3150																						
	Size 2	GS.F3160																						
	Size 3	GS.F3170																						
	Size 4	GS.F3180																						
	Size 5	GS.F3190																						
52. Extraction Forceps GS.1506																								

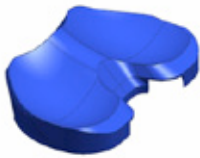53. CR Trial Insert MBH		Size 1	0	GS.T3200	
			+2	GS.T3300	
			+4	GS.T3400	
			+7	GS.T3500	
			0	GS.T3600	
		Size 2	+2	GS.T3700	
			+4	GS.T3800	
			+7	GS.T3900	
			0	GS.T4000	
		Size 3	+2	GS.T4100	
			+4	GS.T4200	
			+7	GS.T4300	
		Size 4	0	GS.T4400	
			+2	GS.T4500	
			+4	GS.T4600	
		Size 5	+7	GS.T4700	
			0	GS.T4800	
			+2	GS.T4900	
				+4	GS.T5000
				+7	GS.T5100

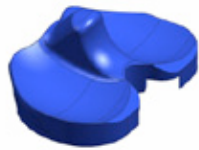55. PS Trial Insert MBH		Size 1	0	GS.T5200	
			+2	GS.T5300	
			+4	GS.T5400	
			+7	GS.T5500	
		Size 2	0	GS.T5600	
			+2	GS.T5700	
			+4	GS.T5800	
			+7	GS.T5900	
		Size 3	0	GS.T6000	
			+2	GS.T6100	
			+4	GS.T6200	
		Size 4	+7	GS.T6300	
			0	GS.T6400	
			+2	GS.T6500	
		Size 5	+4	GS.T6600	
			+7	GS.T6700	
			0	GS.T6800	
				+2	GS.T6900
				+4	GS.T7000
				+7	GS.T7100

54. CR Trial Insert FB		Size 1	0	GS.T10000	
			+2	GS.T10010	
			+4	GS.T10020	
			+7	GS.T10030	
			+10	GS.T10040	
			+13	GS.T10050	
		Size 2	0	GS.T10060	
			+2	GS.T10070	
			+4	GS.T10080	
			+7	GS.T10090	
			+10	GS.T10100	
			+13	GS.T10110	
		Size 3	0	GS.T10120	
			+2	GS.T10130	
			+4	GS.T10140	
			+7	GS.T10150	
			+10	GS.T10160	
		Size 4	+13	GS.T10170	
			0	GS.T10180	
			+2	GS.T10190	
			+4	GS.T10200	
			+7	GS.T10210	
			+10	GS.T10220	
		Size 5	+13	GS.T10230	
			0	GS.T10240	
			+2	GS.T10250	
			+4	GS.T10260	
			+7	GS.T10270	
				+10	GS.T10280
				+13	GS.T10290

56. PS Trial Insert FB		Size 1	0	GS.T10300	
			+2	GS.T10310	
			+4	GS.T10320	
			+7	GS.T10330	
			+10	GS.T10340	
			+13	GS.T10350	
		Size 2	0	GS.T10360	
			+2	GS.T10370	
			+4	GS.T10380	
			+7	GS.T10390	
			+10	GS.T10400	
			+13	GS.T10410	
		Size 3	0	GS.T10420	
			+2	GS.T10430	
			+4	GS.T10440	
			+7	GS.T10450	
			+10	GS.T10460	
		Size 4	+13	GS.T10470	
			0	GS.T10480	
			+2	GS.T10490	
			+4	GS.T10500	
			+7	GS.T10510	
			+10	GS.T10520	
		Size 5	+13	GS.T10530	
			0	GS.T10540	
			+2	GS.T10550	
			+4	GS.T10560	
			+7	GS.T10570	
				+10	GS.T10580
				+13	GS.T10590